



MEGAHILO



**32mg - (L-HA) + 32mg - (H-HA)**





MEGAHILO

MEGAHILO is a tissue stimulator for  
biomodelling and skin profiloplasty.

Thanks to the 5-point passing technique,  
MEGAHILO allows you to restore the correct  
geometry and  
proportions of the face, losing  
with age.



## MEDISEPTIC

The action of the treatment with the use of MEGAHILO is slimming facial features and strong stimulation of fibroblasts because of the high concentration of low molecular weight and high molecular weight hyaluronic acid, the preparation causes multifaceted revitalization of the skin and secondary restoration of the harmony of facial contours.  
ONLY 5 INJECTION POINTS

## RECORD AMOUNT OF HYALURONIC ACID

A hybrid combination of two sizes of acid particles for full performance optimization surgery

32mg - low molecular weight L-HA  
(80-100KDA)

32mg - H-HA with high molecular weight  
(1100-1500KDA)



## HYBRID LOW AND HIGH MOLECULAR HYALURONIC ACID

- Long-term release
- Instant effects
- No BDDE cross-linking
- Short and non-invasive treatment procedure

32mg - L-HA + 32mg - H-HA

## MEDISEPTIC

MEGAHILO injected into the skin naturally increases the volume of the skin and tissue emphasizes the natural contours of the face. Giving volume and smoothing effect results from the high content of bimolecular hyaluronic acid and its ability to revitalize and bind water. The product is gradually resorbable, and the tissue enhancement effect diminishes over time.

ONLY 5 INJECTION POINTS



## PROFILO-PLASTIC SKIN

The skin PROFILO-PLASTIC treatment consists in restoring the volume and correct proportions of the face related to the movement of the "youth triangle". The treatment allows you to emphasize and reconstruct the natural anatomical bulges of the face so that it regains youthful features and shape.

## INTENSIVE BIO-REVITALIZATION

The existence of a massive amount of bimolecular hyaluronic acid in MEGAHILO not only improves the contour and profile of the face immediately but it also strongly revitalizes and rejuvenates the skin.

## TREATMENT INDICATIONS

Profiloplasty of the face, neck, cleavage, and hands Remodeling and bio reconstruction of tissues Restoration of three-dimensional faces Visible lifting effect extreme hydration of tissues increased volume in places of atrophy, a natural volumetric effect.



## FOR PERFECT FACE PROFILE AND YOUTHFUL SKIN

Treatment technique MEGAHILO covers only 5 injections per side. The treatment procedure has been specially developed the designated puncture points were placed in areas with a high lymphatic structure, and the product was rapidly dispersed and redistributed over the entire surface of the skin.

The special application technique also ensures high surgical safety, avoids anatomical places presence of large blood vessels.

By minimizing the number of punctures, the procedure MEGAHILO does not require anesthesia and a return to normal functioning is possible.

Immediately after the procedure.

Pass MEGAHILO does not cause discomfort, swelling, and bruising.



## SUPPLY TECHNIQUE

MEGAHILO is designed for medium injections layers of the skin by specialists in aesthetic medicine trained in implantation techniques hyaluronic acid. Apply in the middle layers of the skin, pointwise, at designated injection sites (APPOINTS).

## MEDISEPTIC

only 5 injection points - MEGAHILO application technique has been developed to minimize the number of punctures and the risk of complications.

- up to 64 mg of hyaluronic acid- an unprecedented amount of hyaluronic acid with high and low molecular weight and selected chain length

- only two treatments - shortest possible treatment series

- no pain and no recovery time - specialized application technique, without causing discomfort, swelling and bruising

- immediate effect - a delicate volumetric effect is visible immediately after the treatment, with the full effect of skin

revitalization appearing after about 4 weeks.





**NEEDLE:**

30G/13mm

**PRODUCT QUANTITY:**


1ml per page

**NUMBER OF TREATMENTS:**

2 treatments (next one after 4 weeks break)

**FREQUENCY:**

2 times a year

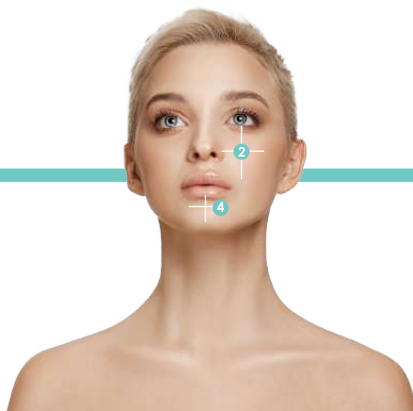


**JUST ABOUT 5 POINTS TO PERFECT PROFILE FACE AND YOUTHFUL SKIN**



### POINT 1

Zygomatic bone outgrowth  
Measure 2 cm from the outer corner of the eye to the side, then 2 cm down.  
Apply 0.2 ml to the measured point.  
MEGAHILO

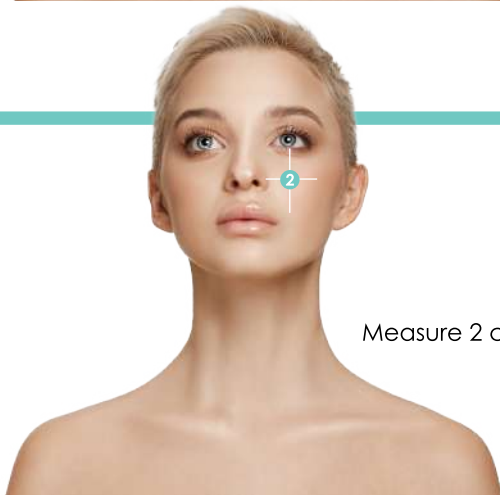

### POINT 2

Nose wing  
Measure 2 cm to the side from the middle cartilaginous part of the wing nose.  
Apply 0.2 ml to the measured point.  
MEGAHILO




### POINT 3

Earlobe

Measure 1 cm to the side from the tip of the ear.

Apply 0.2 ml to the measured point

MEGAHILO

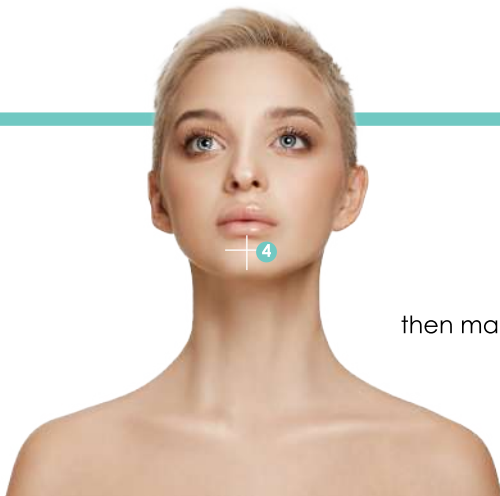

### POINT 4


Chin

Divide the beard vertically into 2 equal parts,  
then mark perpendicular horizontal line at 1/3 of the height of the chin.

Measure 1.5 cm from the intersection of the two lines.

Apply 0.2 ml to the measured point

MEGAHILO

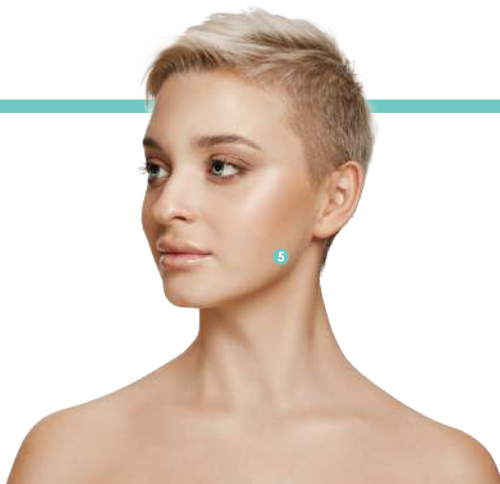

### POINT 5

Jaw

Measure 1 cm up from the angle of the mandible.

Apply 0.2 ml to the measured point.

MEGAHILO



### NEEDLE:

30G / 13MM

### PRODUCT QUANTITY:

2ml for the entire treatment area

### NUMBER OF TREATMENTS:

2 treatments (the next one after a 4-week break)

### FREQUENCY:

2 times a year

**NEEDLE:**

30G / 13MM

**PRODUCT QUANTITY:**

1ml for one hand


**NUMBER OF TREATMENTS:**

2 treatments (the next one after a 4-week break)

**FREQUENCY:**

2 times a year



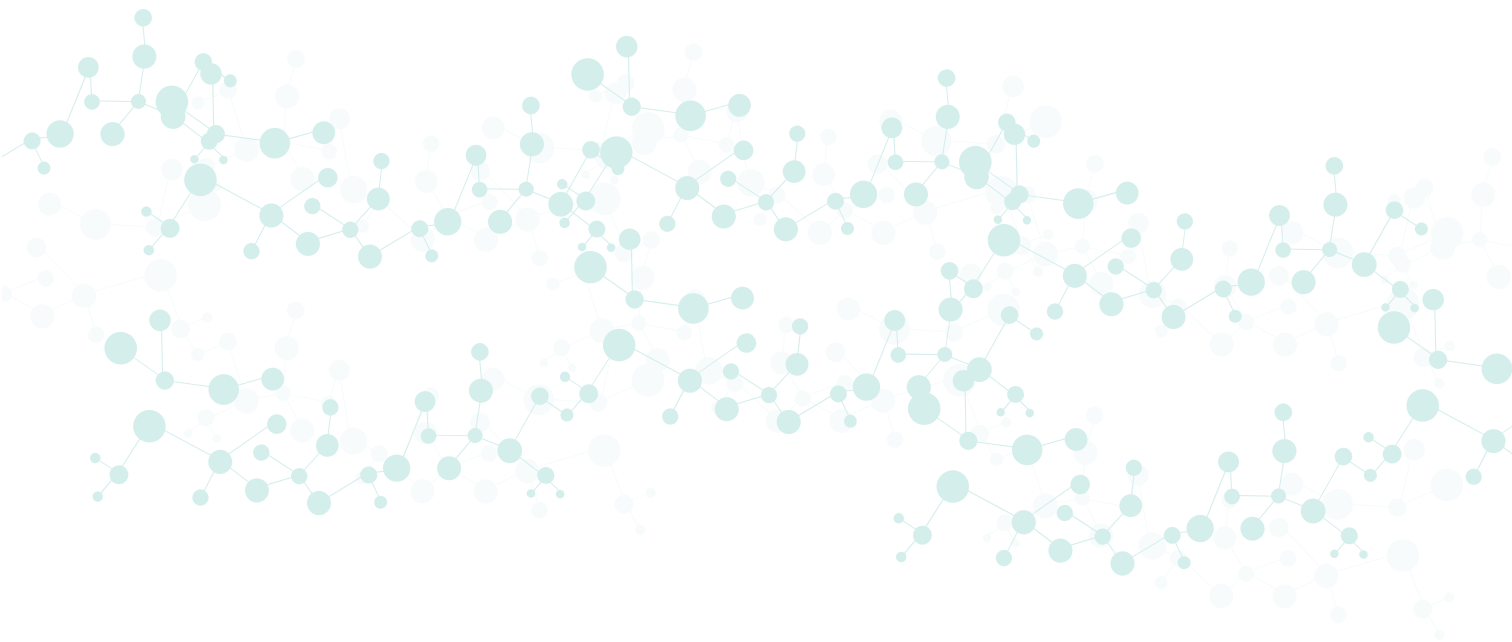

Manufacturer:  
Laboratory Hyamed SA  
#Rue de Lausanne 20BIS, 1201, Geneva, Switzerland